

OrthoSave® Comp Compression Therapy

Patient User Guide

Basic Purpose of the OrthoSave® Comp

The OrthoSave® Comp provides controlled compression therapy for the reduction of swelling and pain as well as increasing joint range of motion.

Your Daily Prescription (default prescription in parentheses)

Use for _____ (30) minutes with 1 hour off between sessions _____ (3) times per day for _____ (16) days.

Basic Patient Guide for Use of the OrthoSave® Comp and OrthoSave® DVT

- Step 1. Place all wraps on appropriate body parts.
- Step 2. Elevate affected body parts if possible or reasonable.
- Step 3. Make sure power is on at the back of the Comp unit.
- Step 4. Press on/off button (front upper right corner of display) until the second beep, then Press ^ (up arrow) to turn on for full display.
- Step 5. When session is completed, press the on/off button until unit shuts off.



Basic Troubleshooting

- ***If needing to stop in mid therapy:***
Press on/off button to turn off. Therapy will stop. Disconnect wraps from your body.
- ***If flow of therapy is blocked:***
Check for folds or kinks in the line and correct them. Disconnect all connections to the wrap and the back of the machine and reconnect to be certain all connections are fitted properly. Restart therapy.

FAULT CODES

These therapy devices are preprogrammed with error messages called faults. Common fault codes:

- *2013: Wrap over-compressed during inflation. Shut unit off then turn back on to reset.*
- *2050: Wrap not properly connected. Check connections then turn unit back on to reset.*
- *2051: Air restriction in line. Check for pinched wrap air lines then turn unit back on to reset.*

For all other questions: Contact _____ at _____.
(Rep Name)