

# OrthoSave® Comp and OrthoSave® DVT Compression Therapies

## Patient User Guide

### Basic Purpose of the OrthoSave® Comp and OrthoSave® DVT Compression Therapy

The OrthoSave® Comp provides controlled iceless compression therapy for the reduction of swelling and pain as well as increasing joint range of motion. In addition the OrthoSave® DVT offers mobile, mechanical blood clot prevention.

### Your Daily Prescription (default prescription in parentheses)

Use for \_\_\_\_\_ (30) minutes with 1 hour off between sessions \_\_\_\_\_ (3) times per day for \_\_\_\_\_ (16) days.

### Basic Patient Guide for Use of the OrthoSave® Comp and OrthoSave® DVT

- Step 1. Place all wraps on appropriate body parts.
- Step 2. Elevate affected body parts if possible or reasonable.
- Step 3. Make sure power is on at the back of the Comp unit.
- Step 4. Press on/off button (front upper right corner of display) until the second beep, then Press ^ (up arrow) to turn on for full display.
- Step 5. When session is completed, press the on/off button until unit shuts off.



### Basic Troubleshooting

- **If needing to stop in mid therapy:**  
Press on/off button to turn off. Therapy will stop. Disconnect wraps from your body.
- **If flow of therapy is blocked:**  
Check for folds or kinks in the line and correct them. Disconnect all connections to the wrap and the back of the machine and reconnect to be certain all connections are fitted properly. Restart therapy.
- **If OrthoSave® DVT will not turn on:**  
Battery charge is low. Plug unit into the OrthoSave® DVT power supply or install into the OrthoSave® Comp unit to continue use. Allow OrthoSave® DVT to charge 3-8 hours before using on battery power.



### FAULT CODES

*These therapy devices are preprogrammed with error messages called faults. Common fault codes:*

- ON OrthoSave® DVT Unit
  - 1013: Can occur if wrap pressed by outside agent during inflation. *Shut unit off then back on to reset.*
  - 1050: Wrap not properly connected or line leak. *Check connections then turn unit back on to reset.*
  - 1051: Air restriction in line. *Check for pinched wrap air lines then turn unit back on to reset.*
- ON OrthoSave® Comp Unit
  - 2013: Wrap over-compressed during inflation. *Shut unit off then turn back on to reset.*
  - 2050: Wrap not properly connected. *Check connections then turn unit back on to reset.*
  - 2051: Air restriction in line. *Check for pinched wrap air lines then turn unit back on to reset.*

**For all other questions:** Contact \_\_\_\_\_ at \_\_\_\_\_.  
(Rep Name)