

Bone Growth Stimulation Therapy

Patient User Guide

Basic Purpose of the Bone Growth Stimulation Therapy

Use combined static and alternating magnetic fields to grow bone by effectively targeting growth factors needed to begin bone formation.



Your Daily Prescription

Use for 30 minutes per day for _____ days.

Basic Patient Guide for Use of Bone Growth Stimulator after delivery

- Step 1. Strap on bone growth stimulator around affected bone.
- Step 2. Turn on the machine. (A slight tingling sensation may be felt during the therapy session).
- Step 3. Turn the machine off when your 30-minute session is completed.

Basic Troubleshooting

- ***If needing to stop in mid therapy:***
Turn machine off. Pull off straps.

- ***If therapy seems not to be starting:***
Make sure batteries are functional and properly placed.

For all other questions: Contact _____ at _____.
(Rep Name)